INADINE®

PVP-I NON ADHERENT DRESSING

EN SCHEDULING STATUS **S0**

PROPRIETARY NAME (AND DOSAGE FORM)

INADINE® non-adherent dressing.

COMPOSITION

A gauze of knitted viscose rayon impregnated with an ointment containing 10% povidone-iodine, providing 0,9% m/m available iodine.

PHARMACOLOGICAL CLASSIFICATION

A.14.2 Wound dressings.

PHARMACOLOGICAL ACTION

Povidone-iodine is an iodophore which slowly liberates inorganic iodine when in contact with the skin. Povidone-iodine has broad spectrum antiseptic activity.

INDICATIONS

INADINE® dressings may be used for prophylaxis of infections in minor skin injuries involving superficial loss.

CONTRA-INDICATIONS

Hypersensitivity to povidone-iodine.
Application to large areas of broken skin (e.g. severe and extensive burns) should be avoided as excessive absorption of iodine may occur.

WARNINGS

Not to be used by persons allergic to iodine.

INTERACTIONS

Absorption of povidone-iodine may interfere with thyroid function tests.

PREGNANCY AND LACTATION

Not to be used during pregnancy or by lactating women.

DOSAGE AND DIRECTIONS FOR USE

The dressing is for external use only.
INADINE® dressings are non-adherent to ensure minimal damage to healing tissues on removal.
The affected areas should be cleansed with an antiseptic preparation and dried.
INADINE® dressings may be applied as often as required.

SIDE-EFFECTS AND SPECIAL PRECAUTIONS

If local irritation, redness or swelling develops, discontinue the use of dressing.

KNOWN SYMPTOMS OF OVERDOSAGE AND PARTICULARS OF ITS TREATMENT

Systemic effects including metabolic acidosis, hypernatremia and renal impairment may follow the application of povidone-iodine to severe burns or large areas otherwise denuded of skin.
Treatment is symptomatic and supportive.

IDENTIFICATION

A cream coloured gauze of knitted viscose rayon impregnated with an orange brown ointment.

PRESENTATION

INADINE® dressings are individually packed in sealed peelable pouches and supplied in cartons of 50 pouches.
Two sizes are available: 5 cm x 5 cm and 9,5 cm x 9,5 cm.

STORAGE INSTRUCTIONS

Store below 25°C.
KEEP OUT OF REACH OF CHILDREN.

REGISTRATION NUMBER

X/14.2/191

NAME AND BUSINESS ADDRESS OF THE HOLDER OF THE CERTIFICATE OF REGISTRATION

Systagenix Wound Management South Africa (Pty) Ltd.
(Reg. No. 2009/010315/07)
Building 24, Thornhill Office Park
94 Bekker Road
Vorna Valley, Midrand,
Gauteng,
1685
South Africa

DATE OF PUBLICATION OF THE PACKAGE INSERT

August, 2013.

Manufacturer
©2015 Systagenix Wound Management Limited,
Gargrave, North Yorkshire, BD23 3RX, U.K.

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