Let's Protect®

---

# ODOLOCK™

ACTIVATED CHARCOAL DRESSING

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STERILE R

 Systagenix

## **EN** Product Description

ODOLOCK™ Activated Charcoal Dressing is a sterile dressing, composed of pure activated carbon encased in a non woven nylon envelope. By eliminating offensive odours ODOLOCK™ Activated Charcoal Dressing improves the quality of life for the patient and nursing personnel.

### **Indications**

ODOLOCK™ Activated Charcoal Dressing is suitable for use in the management of all chronic wounds including fungating carcinomas, diabetic ulcers, decubitus (pressure) ulcers, venous ulcers, arterial ulcers, mixed aetiology ulcers, traumatic and surgical wounds where odour occurs.

### **DIRECTIONS FOR USE**

#### **Preparation of wound site**

Dry, necrotic tissue must first be removed by surgical, enzymatic or autolytic debridement.

#### **Application**

- Prepare the wound according to wound management protocol.
- Prior to application, ODOLOCK™ Activated Charcoal Dressing can be impregnated with saline solution or sterile water.
- The adsorptive properties of the dressing are most fully utilised when ODOLOCK™ Activated Charcoal Dressing makes direct contact with the wound.
- Either side of ODOLOCK™ Activated Charcoal Dressing can be placed on top of the wound site.
- ODOLOCK™ Activated Charcoal Dressing can be easily packed into deep wounds.
- The outer nylon envelope enables removal from the majority of wounds without adherence or trauma.

- If required, ODOLOCK™ Activated Charcoal Dressing can be placed on top of a non-adherent wound contact layer. In this case it is important to ensure that the mechanism of action of ODOLOCK™ Activated Charcoal Dressing is not impaired by excess ointment or grease of the contact layer.
- Depending on the level of exudate an absorbent, secondary dressing may be placed on top of ODOLOCK™ Activated Charcoal Dressing.
- Fix the dressing in place in a manner appropriate for the indication.

### **Changing the dressing**

Dressing change frequency is dictated by good wound care practice and will depend on the condition of the wound; ODOLOCK™ Activated Charcoal Dressing may be left in place for several days, while the secondary absorbent dressing is changed as required.

ODOLOCK™ Activated Charcoal Dressing should not be cut; otherwise particles of activated carbon may get into the wound, and cause discoloration.

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= Sterilized using irradiation.

= Consult instructions for use.

= CE-mark and identification number of notified body.

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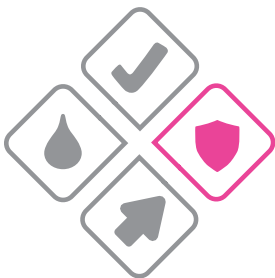
**LET'S HEAL®**

519002S.b

©2018 Systagenix Wound Management Limited,  
Gargrave, North Yorkshire, BD23 3RX, UK

**CE**  
**0086**

2018-06