

ADAPTIC™

DIGIT


NON-ADHERING DIGIT DRESSING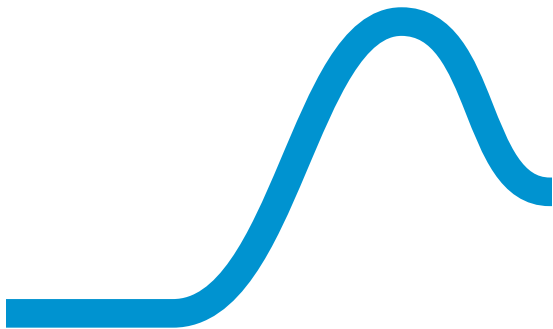


AN ACELITY COMPANY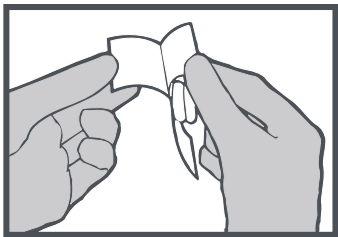

1.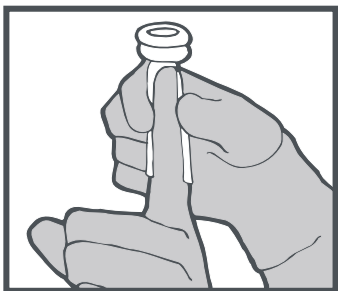

2.

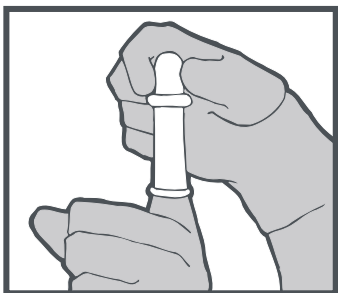3.

EN Product Description

ADAPTIC™ DIGIT Non-Adhering Digit Dressing is made of a unique tacky silicone coated fabric with a bandage that is specifically designed for dressing fingers and toes. The dressing does not adhere to the wound, is conformable and non-bulky, therefore allowing maximum digit flexibility. ADAPTIC™ DIGIT is quick and simple to apply and removes cleanly from the wound, minimizing trauma.

Indications

ADAPTIC™ DIGIT dressing is indicated to manage a range of mild to severe digit injuries.

DIRECTIONS FOR USE

Site Preparation

1. Prepare the wound according to wound management protocol.
2. Ensure skin surrounding the wound is dry.

Dressing Application

1. Remove shiny backing strip (see diagram 1).
2. Place primary non-adhering layer over digit tip and gently smooth to cover any broken skin (see diagram 2).
3. Roll down lower ring of tube bandage over primary strip (and secondary dressing if required). Follow this with the top ring to form a double layered tube for protection (see diagram 3).

Caution: Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner.

Do not re-use (single use).

Do not re-sterilize.

Do not use if package is damaged.

The use by date of this product is printed on the packaging.

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©2018 Systagenix Wound Management Limited,