 Let's Comfort®

ADAPTIC®

NON-ADHERING DRESSING

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PANSEMENT NON-ADHERENT

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ICKE VIDHÄFTANDE FÖRBAND

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TARTTUMATON TAITOS

IKKE-HÆFTENDE FORBINDING

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 Systagenix

EN Product Description

ADAPTIC® Non-Adhering Dressing is a primary dressing made of knitted cellulose acetate fabric and impregnated with a specially formulated petrolatum emulsion. It is designed to help protect the wound while preventing the dressing from adhering to the wound.

- Helps prevent dressing adherence
- Helps protect regenerating tissue and minimizes patient pain and trauma during dressing changes
- Helps prevent the pooling of fluid at the wound site
- Exudate easily passes through to the secondary absorbent dressing
- Can be cut to wound size without unravelling and shredding

Indications

ADAPTIC® Non-Adhering Dressing is indicated for dry to highly exuding wounds where adherence of dressing and exudate is to be prevented including:

- 1st & 2nd degree burns
- Abrasions
- Grafts
- Venous ulcers
- Pressure ulcers
- Nail extractions
- Eczema
- Staples
- Surgical incisions
- Lacerations
- Reconstructive procedures
- Suture lines

DIRECTIONS FOR USE

Site Preparation

1. Prepare the wound according to wound management protocol.
2. Ensure skin surrounding the wound is dry.

Dressing Application

1. Carefully clean the wound following facility protocol.
2. Cut ADAPTIC® Dressing to size if required and place directly on the wound.
3. Cover with sterile secondary dressing of your choice depending on the amount of wound exudate.
4. ADAPTIC® Dressing may be left in place for several days depending upon amount of exudate.

Do not use if package is damaged.

Do not re-use (single use).

Do not resterilise.

The use by date of this product is printed on the packaging.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner.

This caution is not applicable outside the U.S.

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©2018 Systagenix Wound Management Limited,