BIOCLUSIVE™ PLUS

TRANSPARENT FILM DRESSING

TRANSPARENTER FILMVERBAND

PANSEMENT-FILM TRANSPARENT

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Acelity™

EN PRODUCT DESCRIPTION

BIOCLUSIVE™ PLUS Transparent Film Dressing is a self-adhesive dressing consisting of a transparent semi-permeable polyurethane film coated with an acrylic adhesive.

BIOCLUSIVE™ PLUS provides a bacterial barrier while also allowing the transmission of moisture vapour and oxygen, and facilitates a moist wound healing environment. BIOCLUSIVE™ PLUS allows visual inspection of the wound site and is flexible and conformable for use on difficult contours.

INDICATIONS

BIOCLUSIVE™ PLUS dressing is indicated for the management of wounds where there is no exudate, or light levels of exudate.

BIOCLUSIVE™ PLUS dressing should be used under health care professional direction for the following indications:

- Minor burns
- Donor sites
- Superficial pressure areas and leg ulcers
- Clean, closed, post-operative wounds
- Cuts and abrasions

BIOCLUSIVE™ PLUS is also suitable for:

- Securing of catheters
- Use as a secondary, cover dressing
- Reducing shear and friction

Contra-indications:

BIOCLUSIVE™ PLUS dressing should not be used on patients with a known sensitivity to any of its components.

BIOCLUSIVE™ PLUS dressing is not indicated for use on the following:

- Full-thickness burns
- Deep cavity wounds
- Wounds or catheter sites which are clinically infected
- Wounds treated with topical medicinal preparations

Precautions

If clinical infection is observed during the use of BIOCLUSIVE™ PLUS dressing, discontinue use and refer to local protocol for managing the infection.

The bacterial barrier of the dressing can be compromised if the dressing is damaged or punctured. Leakage or loss of adhesion to skin will also compromise the bacterial barrier.

DIRECTIONS FOR USE

PREPARATION:

1. Prepare the wound according to local wound management protocol.
2. Ensure that the skin surrounding the wound is clean, dry and free from grease, soaps or detergents.

DRESSING APPLICATION:

1. Choose an appropriate size dressing to allow at least a 2cm border to overlap the wound edge.
2. Peel open the package and remove the dressing.
3. Remove the larger, central release liner from the adhesive underside of the dressing (marked A).
4. Position the dressing centrally over the wound, adhesive side down.
5. Press the exposed adhesive side of the dressing onto the skin surrounding the wound.
6. Grasp the outer release liner tabs from the underside of the dressing (marked B).
7. Peel away the tabs and press the remaining edges of the dressing onto the skin using a rolling motion.
8. Ensure the dressing adheres well to the skin.
9. Once the dressing is secure, grasp the upper release liner tabs (marked C) and peel away from the centre of the dressing outwards.

DRESSING change and removal

- Dressing change frequency is dictated by good wound care practice and will depend on the condition of the wound.
- BIOCLUSIVE™ PLUS dressing may be left in place for several days depending upon amount of exudate.

To remove BIOCLUSIVE™ PLUS from the wound, gently peel and lift one corner of the dressing from the skin. Support the skin whilst peeling the dressing off by stretching horizontally (not vertically) and in the direction of hair growth.

Care should be taken to avoid skin damage with repeated applications or on patients with fragile skin. Care should also be taken not to dislodge catheters or other devices.

Use existing clinical protocols to clean the wound in order to remove any remaining exudate residue before wound assessment or the application of further dressings.

Do not use if package is damaged.

Do not re-use.

Do not re-sterilize.

The use by date of this product is printed on the packaging.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner. This caution is not applicable outside the U.S.

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©2016 Systagenix Wound Management Limited,