



LET'S HEAL®



Let's Comfort®

ADAPTIC TOUCH®

NON-ADHERING SILICONE DRESSING

STERILE R

 Systagenix

Product Description

ADAPTIC TOUCH® Non-Adhering Silicone Dressing is a flexible, open-mesh primary wound contact layer comprised of cellulose acetate coated with a soft tack silicone. The soft tack silicone assists dressing application, prevents adherence of the secondary dressing to the wound and is atraumatic to the wound and surrounding skin. The open mesh structure allows free passage of exudate into an absorbent secondary dressing. ADAPTIC TOUCH® Non-Adhering Silicone Dressing helps protect the wound while minimizing patient pain and trauma on removal and at dressing change.

Indications

ADAPTIC TOUCH® Non-Adhering Silicone Dressing is designed as a primary wound contact layer for use in the management of dry to heavily exuding, partial and full-thickness chronic wounds including venous ulcers, decubitus (pressure) ulcers and diabetic ulcers, and for traumatic and surgical wounds, donor sites and 1st and 2nd degree burns.

Contraindications

ADAPTIC TOUCH® Non-Adhering Silicone Dressing is not indicated for:

- Patients with a known sensitivity to silicone or cellulose acetate fabric.
- Surgical implantation.

Precautions

- To prevent potential adherence of silicone to gloves, gloves may be moistened with a sterile solution to facilitate handling.
- The wound should be inspected for signs of infection according to clinical practice.

Dressing selection

- Select a size of ADAPTIC TOUCH® that is larger than the

wound to ensure dressing can be applied to intact skin surrounding the wound margins.

- ADAPTIC TOUCH® can be cut to size, if necessary. Dressing will not unravel or shed fibres.
- If more than one piece of ADAPTIC TOUCH® is required, ensure dressings overlap, to avoid secondary dressing adherence to the wound. Overlap should be minimized to prevent occlusion of holes.

Dressing application

- Prepare the wound according to wound management protocol.
- Ensure skin surrounding the wound is dry.
- Remove protective films from dressing.
- Place ADAPTIC TOUCH® dressing directly over the wound and smooth in place around the wound.
- Cover with an appropriate absorbent secondary dressing, e.g. TIELLE® Hydropolymer Dressing.

ADAPTIC TOUCH® may also be suitable for use, under medical supervision, with negative pressure wound therapy (NPWT) for the protection of fragile wound structures.

Dressing Change

- Dressing change frequency is dictated by good wound care practice and will depend on the condition of the wound.
- ADAPTIC TOUCH® may be left in place for several days.

Do not use if package is damaged.

The use by date of this product is printed on the packaging.

Do not re-use/re-sterilize.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner.

This caution is not applicable outside the U.S.

SYMBOLS USED ON LABELLING



= Do not use if package is damaged.



= Do not re-use (single use).



= Do not re-sterilize.



= Sterile. Method of sterilization: irradiation.



= Consult instructions for use.



= Caution.



= Store below 30°C (86°F).



= Batch number.



= Use by: year and month.



= Catalogue Number.



= See instructions for use for exudate level.

518035S.e



Manufacturer

©2014 Systagenix Wound Management Limited,
Gargrave, North Yorkshire, BD23 3RX, U.K.

