## V.A.C.® GEL INSTRUCTIONS FOR USE AND SAFETY INFORMATION

### KEEP THERAPY ON

Never leave negative pressure off for more than 2 hours per day. Remove V.A.C.® dressing if negative pressure is terminated or is off for more than 2 hours per day.

### DRESSING CHANGES

Perform aggressive wound cleaning per physician order prior to dressing application. Routine dressing changes should occur every 48 hours. Dressing changes for infected wounds should be accomplished every 12-24 hours. Always replace with sterile disposables from unopened packages. Follow established institution protocols regarding clean versus sterile technique.

**NOTE:** *All components of the V.A.C.® System are packaged sterile. The decision to use clean versus sterile/ aseptic technique is dependent upon wound pathophysiology and physician/clinician preference. All components of V.A.C.® Therapy disposables including the foam, canister, tubing and drape are latex free.*

### MONITORING THE WOUND

Inspect the dressing frequently to ensure foam is collapsed and negative pressure is being delivered in a consistent manner. Monitor periwound tissue and exudate for signs of infection\* or other complications. Extra care and attention should be given if there are any signs of possible infection or related complications. Infection can be serious. With or without V.A.C.® Therapy, infection can lead to many adverse complications including pain, discomfort, fever, gangrene, toxic shock, septic shock and various other complications.

### IF DRESSING ADHERES TO WOUND

Instill normal saline into the dressing and let it set for 15-30 minutes, then gently remove the dressing from the wound. Consider placing a single-layer, wide-meshed, non-adherent dressing prior to foam placement.

### DISCOMFORT

If patient complains of discomfort throughout therapy, consider changing to white PVA VersaFoam™. If patient complains of discomfort during the dressing change, consider premedication, use of a non-adherent prior to foam placement or instillation of a topical anesthetic agent such as 1% lidocaine without epinephrine prior to dressing removal.

### UNSTABLE STRUCTURES

Over unstable body structures such as unstable chest wall or non-intact fascia, use continuous (not intermittent) therapy to minimize movement and stabilize the wound bed.

### SPINAL CORD INJURY

In the event a patient experiences autonomic hyperreflexia (sudden elevation in blood pressure or heart rate in response to stimulation of the sympathetic nervous system) discontinue V.A.C.® Therapy to help minimize sensory stimulation.

### BODY CAVITY WOUNDS

Underlying structures must be covered by natural tissues or synthetic materials that form a complete barrier between the underlying structures and the V.A.C.® foam.

### V.A.C.® DRESSING USE

This and all V.A.C.® Dressings distributed by KCI are to be used exclusively with V.A.C.® devices.

**WARNING: Do not pack the foam into any areas of the wound. Forcing foam dressings in a compressed manner into any wound is contrary to KCI recommendations.**

\*Signs of possible infection may include fever, tenderness, redness, swelling, itching, rash, increased warmth in the wound area, purulent discharge or a strong odor. Nausea, vomiting, diarrhea, headache, sore throat with swelling of the mucous membrane, disorientation, high fever (>102F; 38.8 °C), refractory hypotension, orthostatic hypotension, or erythroderma (a sunburn-like rash) may be added signs of more serious complications of infection.

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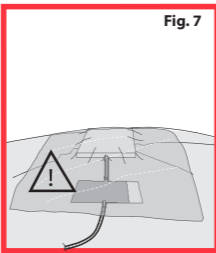
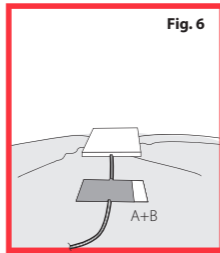
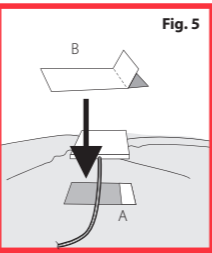
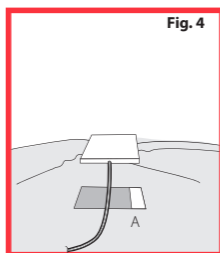
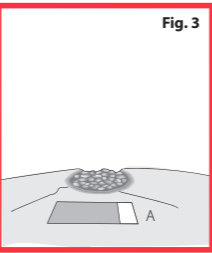
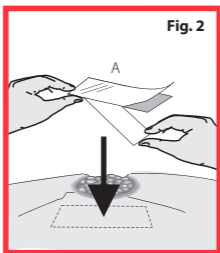
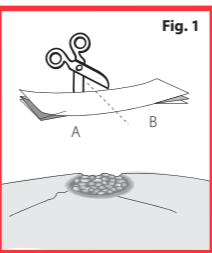
**EU Representative:**  
KCI Medical Products (UK), Ltd.  
11 Nimrod Way  
Wimborne Dorset  
BH21 7SH  
United Kingdom  
www.kci-medical.com

Trademarks designated herein are proprietary to KCI Licensing, Inc. and its affiliates. Trademarks designated \*\* are registered in at least one country where this product is commercialized, but not necessarily in all such countries. The V.A.C.® System and certain components and methods are subject to one or more of the following patents: USA – 4969880, 5100396, 5261893, 5527293, 5636643, 5645081, 6117111, 6142982, 6345623, 6398767, 6553998, 6814079; EU – EP777504, EP688189, EP620720, EP865304, EP465601, EP1088569, EP853950, EP1018967, EP1219311, EP1169071; other patents pending.

Copyright 2017 KCI Licensing, Inc. All rights reserved. 320283 Rev C 11/2017

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KCI USA, Inc.